


CE₁₄₃₄

COVID-19 (SARS-CoV-2) Antigen test kit (Self-Testing)

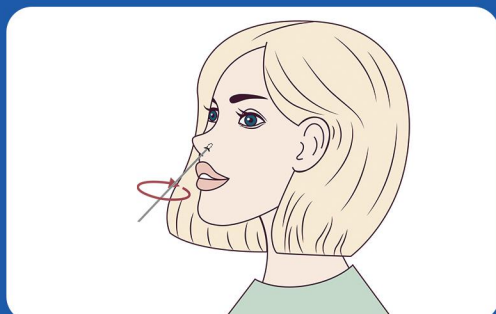

Specification	1 pcs/box 5 pcs/box 25 pcs/box
Specimen	Human Anterior Nasal Swab
Storage	4~30°C

PERFORMANCE

SENSITIVITY: 96.4%(95%CI: 90.8%-98.2%)

SPECIFICITY: 99.8%(95%CI: 94.4%-99.9%)



PRODUCT FEATURES

- ◆ Room temperature storage.
- ◆ No need instrument, get results within 15 minutes.
- ◆ Identify acute or early infection.
- ◆ No reduction in sensitivity test against the UK, South African, Brazilian and Delta variant.



EU HSC mutual recognition (RAT)

UK GOVERNMENT VALIDATED

The UK Government Public Health England, joint PHE Porton Down and University of Oxford was independently evaluated over 140 lateral flow devices that have been referred by the Department of Health and Social Care (DHSC) . Only a few can passed the phase 3A trials, and our DEEPBLUE even have passed phase 3B. That means our test has very high accuracy at multiple viral loads and able to detect the asymptomatic patients and the new different variants.

Easy to use


Using a simple nasal swab within 2cm of the nose makes it extremely easy to administer.



Scan the following QR code to watch the demonstration video on YouTube.



Your kit contains the following materials

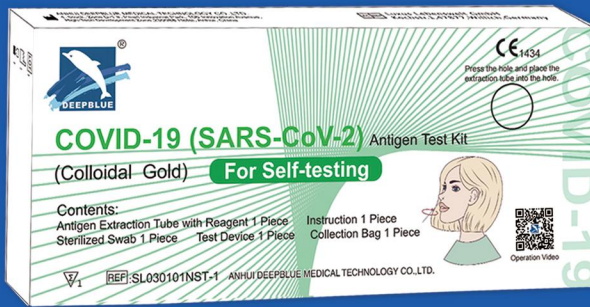

option 1 option 2



Test device

Antigen Extraction Tube


IFU

Nasal swab

Waste bag


TEST PROCEDURE

1. Specimen Collection

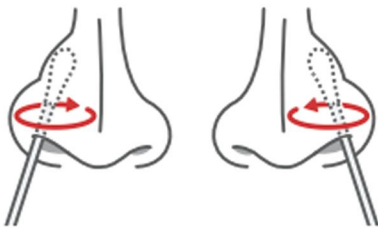


2. Specimen Preparation


option 1

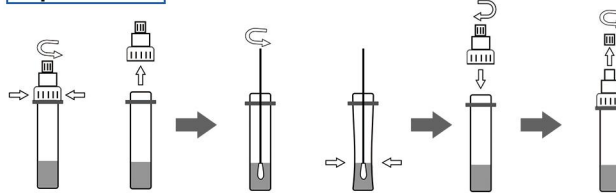

option 2


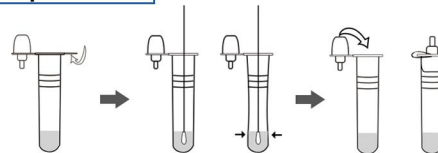


3. Testing

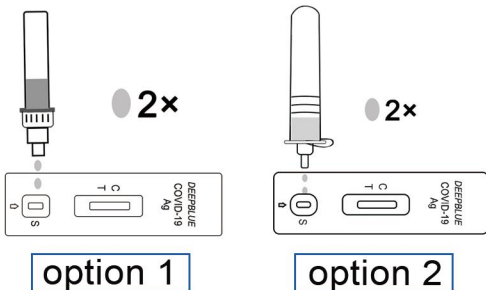

Hold the extraction tube vertically and add two drops of the test specimens into the specimen well (s). Start the timer. Interpret the results at 15 minutes, and the results after 30 minutes are no longer valid.

4. Interpretation of test results

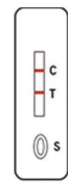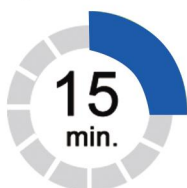

Positive




Negative



Invalid



Invalid

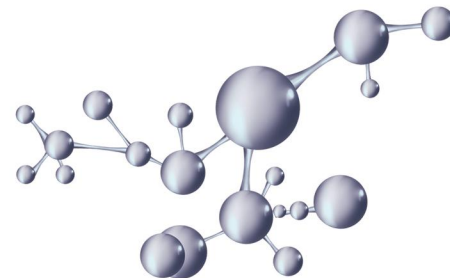

ANHUI DEEPBLUE MEDICAL TECHNOLOGY CO.,LTD.

【Address】 4th, Floor, D-1# Zone, Pearl Industrial Park, 106 Innovation Avenue, High-Tech Development Zone, Hefei 230088, Anhui, China

【Website】 www.dbluemedical.com 【Contact】 0551-65326797