DE EN FR IT

Produktinformationen / Product information
 Information sur le produit / Informazioni sul prodotto

siegmund[®]

FFP2 Atemschutzmaske / Particle filtering half mask
 Masque filtrant jetable / Facciale filtrante monouso

CE 2834
 EN 149:2001+A1:2009 FFP2 NR

Produktname	FFP2 Atemschutzmaske
Product name / Type de produit / Tipo di prodotto	FFP2 Particle filtering half mask / Masque filtrant jetable FFP2 / Facciale filtrante monouso FFP2
Modell / Model / Modèle / Modello	1206
Verordnung	(EU) 2016/425
Regulation / Réglementation / Regolamento	
Material	53,22% Vliesstoff + 23,39% Meltblown Vlies + 23,99% Heißluftbaumwolle
Material / Matériel / Materiali	53,22% Non-woven fabric + 23,39% Meltblown fabric + 23,99% Hot air cotton / 53,22% de non-tissé + 23,39% de tissu Meltblown + 23,99% Coton à air chaud / 53,22% Tessuto non tessuto + 23,39% Tessuto meltblown + 23,99% Cotone ad aria calda
Gültigkeitszeitraum	3 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat
Validity period / Durée de validité / Durata della validità	3 years after production date, see certificate inside / 3 ans après la date de production, voir certificat à l'intérieur / 3 anni dalla data di produzione, vedere il certificato all'interno
Artikelnummer	3065
Article number / Numéro de l'article / Numero dell'articolo	
Prüfnorm	EN 149:2001+A1:2009 FFP2 NR
Test standard / Norme de contrôle/ Standard di prova	
PPE Ausstellende Behörde	CCQS Certification Services Limited
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
Adresse der ausstellenden Behörde	Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin 15, D15 AAK1 - IRELAND
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
Notifizierte Stelle	NB2834
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
Import in die EU & Vertrieb	Siegmund Care GmbH
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
Hersteller	Yantai Lisong Safety Technology Co., LTD
Manufacturer / Fabricant / Fabbricante	Building 2, Huari Industrial Park, No.16, Hongkong Road, Yantai Economic and Technological Development Zone, Yantai City, Shandong Province, China

DE EN FR IT

Produktinformationen / Product information Information sur le produit / Informazioni sul prodotto

siegmund®

DE Warnhinweise

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EN Warnings

1. Before using the masks verify first that the mask is suitable for the intended use. Prior to wearing the user must be adequately trained. It needs to be ensured that the masks are fitting properly.
2. The mask is disposable and cannot be reused.
3. When using the mask with facial hair, the mask may not fit completely dense.
4. The mask needs to be discarded and replaced if it is damaged or shows increased breathing resistance while using.
5. Do not use the mask if there are explosive gases, in highly polluted or deprived environments, in case of fire or for underwater work.
6. Patients with heart disease or other illnesses should remove the mask if they feel unwell.
7. Masks marked "NR" shall not be used for more than one shift.
8. This product does not supply oxygen, use only in adequately ventilated areas containing sufficient oxygen to support life.
9. Don't use this mask when oxygen concentration is less than 19.5%.
10. Do not use in explosive atmosphere.
11. If packaging is damaged the use is strictly prohibited.

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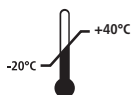
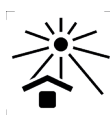
EN Instruction Manual

1. Face the side of the mask without nose clip, then pull the ear loops with two hands to make sure that the nose clip is on the above.
2. Put mask below the chin.
3. After pulling the ear loop to the ear, adjust the ear loop to make yourself comfortable.
4. Place your fingers on the center of the metal nose clip and move your fingers to the sides along the nose clip accordingly until the nose clip is completely in the shape of your nose bridge. Caution! Adjusting the nosepiece with just one hand can cause the mask to leak.
5. Before entering the working area, the user must check if the mask is tightly fitting:
 - I) Cover the mask with two hands to fix the position of the mask on the face.
 - II) If air is leaking from the nose bridge, please readjust the nose clip according to step 4. If air is leaking from the edge of the mask, please readjust the ear loop. If the air leaking problem still cannot be solved, please repeat step 1-4.
 - III) If there is no leakage, you can continue to wear it.

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