DE EN FR IT

Produktinformationen / Product information
Information sur le produit / Informazioni sul prodotto

siegmund®

FFP2 Atemschutzmaske / Particle filtering half mask Masque filtrant jetable / Facciale filtrante monouso

CE 2163
EN 149:2001+A1:2009 FFP2 NR

Produktname	FFP2 Atemschutzmaske
Product name / Type de produit / Tipo di prodotto	FFP2 Particle filtering half mask / Masque filtrant jetable FFP2 / Facciale filtrante monouso FFP2
Modell	JFM02
Model / Modèle / Modello	
Verordnung	(EU) 2016/425
Regulation / Réglementation / Regolamento	
Material	61,5% Spin-Vliesstoff + 38,5% Meltblown Vlies
Material / Matériel / Materiali	61,5% Non-woven fabric + 38,5% Meltblown fabric / 61,5% de non-tissé Spunbond + 38,5% de tissu Meltblown / 61,5% Tessuto non tessuto spunbond + 38,5% Tessuto meltblown
Gültigkeitszeitraum	2 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat
Validity period / Durée de validité / Durata della validità	2 years after production date, see certificate inside / 2 ans après la date de production, voir certificat à l'intérieur / 2 anni dalla data di produzione, vedere il certificato all'interno
Artikelnummer	3190
Article number / Numéro de l'article / Numero dell'articolo	
Prüfnorm	EN 149:2001+A1:2009 FFP2 NR
Test standard / Norme de contrôle/ Standard di prova	
PPE Ausstellende Behörde	UNIVERSAL CERTIFICATION and SURVEILLANCE SERVICES Trade Co.
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
Adresse der ausstellenden Behörde	Necip Fazil Bulvari Keyap Sitesi E2 Blok No:44/84 Yukari Dudullu Ümraniye/ISTANBUL - TURKEY
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
Notifizierte Stelle	NB2163
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
Import in die EU & Vertrieb	Siegmund Care GmbH
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
Hersteller	JIFA GROUP CO., LTD.
Manufacturer / Fabricant / Fabricante	Jiangnan Development Area, 322121 Dongyang, Zhejiang, P.R. China

DE EN FR IT

Produktinformationen / Product information Information sur le produit / Informazioni sul prodotto

siegmund®

DE Warnhinweise

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EN Warnings

1. Before using the masks verify first that the mask is suitable for the intended use. Prior to wearing the user must be adequately trained. It needs to be ensured that the masks are fitting properly.
2. The mask is disposable and cannot be reused.
3. When using the mask with facial hair, the mask may not fit completely dense.
4. The mask needs to be discarded and replaced if it is damaged or shows increased breathing resistance while using.
5. Do not use the mask if there are explosive gases, in highly polluted or deprived environments, in case of fire or for underwater work.
6. Patients with heart disease or other illnesses should remove the mask if they feel unwell.
7. Masks marked "NR" shall not be used for more than one shift.
8. This product does not supply oxygen, use only in adequately ventilated areas containing sufficient oxygen to support life.
9. Don't use this mask when oxygen concentration is less than 19.5%.
10. Do not use in explosive atmosphere.
11. If packaging is damaged the use is strictly prohibited.

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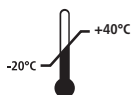
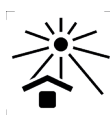
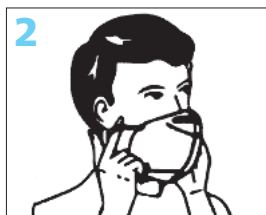
EN Instruction Manual

1. Face the side of the mask without nose clip, then pull the ear loops with two hands to make sure that the nose clip is on the above.
2. Put mask below the chin.
3. After pulling the ear loop to the ear, adjust the ear loop to make yourself comfortable.
4. Place your fingers on the center of the metal nose clip and move your fingers to the sides along the nose clip accordingly until the nose clip is completely in the shape of your nose bridge. Caution! Adjusting the nosepiece with just one hand can cause the mask to leak.
5. Before entering the working area, the user must check if the mask is tightly fitting:
 - I) Cover the mask with two hands to fix the position of the mask on the face.
 - II) If air is leaking from the nose bridge, please readjust the nose clip according to step 4. If air is leaking from the edge of the mask, please readjust the ear loop. If the air leaking problem still cannot be solved, please repeat step 1-4.
 - III) If there is no leakage, you can continue to wear it.

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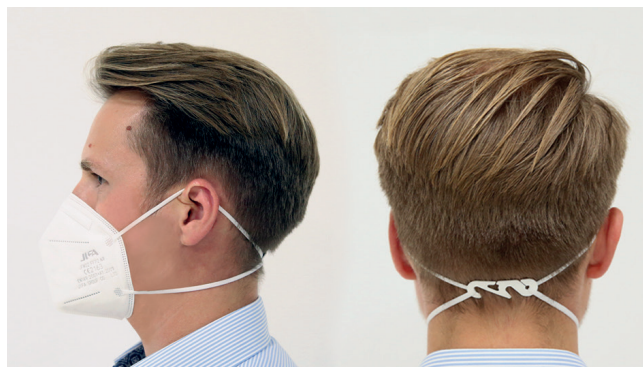
EN Instruction Manual

1. Face the side of the mask without nose clip, then pull the ear loops with two hands to make sure that the nose clip is on the above.
2. Take the enclosed clip and hang it with the more closed side in one ear loop.
3. Put mask below the chin.
4. After you have put the ear loops around your ears, bring the two ear loops together at the lower back of the head and hang the ear loop on the free hook of the clip.
5. Place your fingers on the center of the metal nose clip and move your fingers to the sides along the nose clip accordingly until the nose clip is completely in the shape of your nose bridge. Caution! Adjusting the nosepiece with just one hand can cause the mask to leak.
6. Before entering the working area, the user must check if the mask is tightly fitting: I) Cover the mask with two hands to fix the position of the mask on the face. II) If air is leaking from the nose bridge, please readjust the nose clip according to step 4. If air is leaking from the edge of the mask, please readjust the ear loop. If the air leaking problem still cannot be solved, please repeat step 1-4. III) If there is no leakage, you can continue to wear it.

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