DE EN FR IT

Produktinformationen / Product information
Information sur le produit / Informazioni sul prodotto

siegmund®

FFP2 Atemschutzmaske / Particle filtering half mask Masque filtrant jetable / Facciale filtrante monouso

CE 0082
EN 149:2001+A1:2009 FFP2 NR

Produktname	FFP2 Atemschutzmaske
Product name / Type de produit / Tipo di prodotto	FFP2 Particle filtering half mask / Masque filtrant jetable FFP2 / Facciale filtrante monouso FFP2
Modell	SJ2278
Model / Modèle / Modello	
Verordnung	(EU) 2016/425
Regulation / Réglementation / Regolamento	
Material	50% Heißluftbaumwolle + 30% Meltblown Vlies + 20% Vlies
Material / Matériel / Materiali	50% Hot air cotton + 30% Melt-blown fabric + 20% Non-woven fabrics / 50% Coton à air chaud + 30% de tissu Meltblown+ 20% de non-tissé / 50% Cotone ad aria calda ES + 30% Tessuto meltblown + 20% Tessuto non tessuto
Gültigkeitszeitraum	2 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat
Validity period / Durée de validité / Durata della validità	2 years after production date, see certificate inside / 2 ans après la date de production, voir certificat à l'intérieur / 2 anni dalla data di produzione, vedere il certificato all'interno
Artikelnummer	3064
Article number / Numéro de l'article / Numero dell'articolo	
Prüfnorm	EN 149:2001+A1:2009 FFP2 NR
Test standard / Norme de contrôle/ Standard di prova	
PPE Ausstellende Behörde	APAVE SUDEUROPE SAS
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
Adresse der ausstellenden Behörde	8 rue Jean-Jacques Vernazza – ZAC. Saumaty-Séon – BP 193 13322 Marseille Cedex 16 – France
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
Notifizierte Stelle	NB0082
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
Import in die EU & Vertrieb	Siegmund Care GmbH
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
Hersteller	Jinhua City Shengjie Labor Products Factory
Manufacturer / Fabricant / Fabricante	Shankoufeng Chisong Town, Jinhua City, Zhejiang Province, China

DE EN FR IT

Produktinformationen / Product information Information sur le produit / Informazioni sul prodotto

siegmund®

DE Warnhinweise

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EN Warnings

1. The respirator is disposable, please change within 8 hours.
2. Make sure you wear the respirator properly and rightly.
3. Dispose properly after usage.
4. Please stop using the product if there is any discomfort.
5. Please ensure the mask is fitting tightly before entering contaminated area.
6. Please store out of childrens reach.
7. Avoid direct sunlight, solvents and frost condition during storage and transportation.
8. Please follow relevant health guideline or regulations while using the product.
9. Please read instruction carefully before using the product.

It can offer protection from firm and fluid deleterious kinds of dust, smoke, and aerosols. The particles may be fibrogenic – which means they irritate the respiratory system in the short term and can result in reduction of elasticity of pulmonary tissue in the long run. The total leakage may amount to a maximum of 11%. OEL transgression to the tenfold value.

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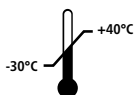
EN Instruction Manual

1. Cup the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hand.
2. Position the respirator under your chin with the nosepiece up.
3. Pull the top strap over your head resting it high at the top back of your head. Pull the bottom strap over your head and position it around the neck below the ears.
4. Place the fingertips of both hands at the top of the metal nosepiece. Mold the nosepiece to the shape of your nose by pushing inward while moving your fingertips down to both sides of the nosepiece. Pinching the nosepiece using one hand may result in less effective respirator performance.
5. The seal of the respirator on the face should be fit-checked prior to wearing in the work area.
 - a) Cover the front of the respirator with both hands. Being careful not to disturb the position of respirator.
 - b) Inhale sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, adjust position of respirator and/or tension of strap. Retest the seal. Repeat the procedure until the respirator is sealed properly.

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