DE EN FR IT

Produktinformationen / Product information
Information sur le produit / Informazioni sul prodotto

siegmund®

FFP3 Atemschutzmaske / Particle filtering half mask Masque filtrant jetable / Facciale filtrante monouso

CE 2834
EN 149:2001+A1:2009 FFP3 NR

Produktname	FFP3 Atemschutzmaske
Product name / Type de produit / Tipo di prodotto	FFP3 Particle filtering half mask / Masque filtrant jetable FFP3 / Facciale filtrante monouso FFP3
Modell / Model / Modèle / Modello	JFM04
Verordnung / Regulation / Réglementation / Regolamento	(EU) 2016/425
Material	17,80% Vliesstoff (PP) + 15,59% Meltblown Vlies (PP) + 4,10% Nasenbügel + 19,69% Kopfschlaufen + 26,42% Clip Kopfschlaufen + 16,41% Formschaum
Material / Matériel / Materiali	17,80% non-woven fabric (PP) + 15,59% meltblown non-woven (PP) + 4,10% nose clip + 19,69% head loops + 26,42% clip head loops + 16,41% molded foam / 17,80% de non-tissé (PP) + 15,59% de tissu Meltblown (PP) + 4,10% pince nasale + 19,69% serre-tête + 26,42% clip d'ajustage pour serre-tête + 16,41% mousse moulée / 17,80% Tessuto non tessuto (PP) + 15,59% Tessuto meltblown (PP) + 4,10% Clip nasale + 19,69% Imbracatura per la testa + 26,42% Clip per imbracatura per la testa + 16,41% Schiuma
Gültigkeitszeitraum	2 Jahre nach Produktionsdatum, siehe innenliegendes Zertifikat
Validity period / Durée de validité / Durata della validità	2 years after production date, see certificate inside / 2 ans après la date de production, voir certificat à l'intérieur / 2 anni dalla data di produzione, vedere il certificato all'interno
Artikelnummer	3380
Article number / Numéro de l'article / Numero dell'articolo	
Prüfnorm / Test standard / Norme de contrôle / Standard di prova	EN 149:2001+A1:2009 FFP3 NR
PPE Ausstellende Behörde	CCQS Certification Services Limited
PPE issuing authority / Autorité de délivrance DPI / Autorità emittente DPI	
Adresse der ausstellenden Behörde	Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin 15, D15 AAK1 - IRELAND
Address of issuing authority / Adresse de l'autorité émettrice / Indirizzo dell'autorità emittente	
Notifizierte Stelle	NB2834
Notified Body / Organisme d'inspection notifié / Organismo di controllo notificato	
Import in die EU & Vertrieb	Siegmund Care GmbH
Import to the EU & Sales / Importateur et distributeur dans l'UE / Importatore & Distributore nelle EU	Landsberger Straße 180, 86507 Oberottmarshausen, Deutschland / Germany / Allemagne / Germania, Telefon: (+49) 0 82 33 / 84 89 19 - 0, Internet: www.siegmund.care
Hersteller	JIFA GROUP CO., LTD.
Manufacturer / Fabricant / Fabbicante	Jiangnan Development Area, 322121 Dongyang, Zhejiang, P.R. China

DE EN FR IT

Produktinformationen / Product information Information sur le produit / Informazioni sul prodotto

siegmund®

DE Warnhinweise

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EN Warnings

1. Before using the masks verify first that the mask is suitable for the intended use. Prior to wearing the user must be adequately trained. It needs to be ensured that the masks are fitting properly.
2. The mask is disposable and cannot be reused.
3. When using the mask with facial hair, the mask may not fit completely dense.
4. The mask needs to be discarded and replaced if it is damaged or shows increased breathing resistance while using.
5. Do not use the mask if there are explosive gases, in highly polluted or deprived environments, in case of fire or for underwater work.
6. Patients with heart disease or other illnesses should remove the mask if they feel unwell.
7. Masks marked "NR" shall not be used for more than one shift.
8. This product does not supply oxygen, use only in adequately ventilated areas containing sufficient oxygen to support life.
9. Don't use this mask when oxygen concentration is less than 19.5%.
10. Do not use in explosive atmosphere.
11. If packaging is damaged the use is strictly prohibited.

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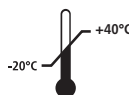
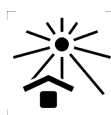
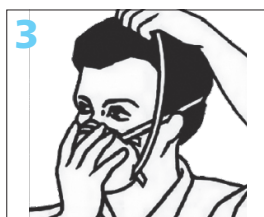
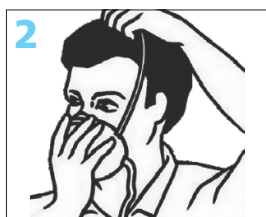
EN Instruction Manual

- Open the fastener of the lower head loop.
- Cup the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hand.
- Position the respirator under your chin with the nosepiece up.
- Pull the top strap over your head resting it high at the top back of your head. Take the two parts of the lower head loop and fasten the fastener at the lower back of the head. You can re-adjust the fit of the mask using the ends of the lower head loop.
- Place the fingertips of both hands at the top of the metal nosepiece. Mold the nosepiece to the shape of your nose by pushing inward while moving your fingertips down to both sides of the nosepiece. Pinching the nosepiece using one hand may result in less effective respirator performance.
- The seal of the respirator on the face should be fit-checked prior to wearing in the work area.
 - Cover the front of the respirator with both hands. Being careful not to disturb the position of respirator.
 - Inhale sharply. A negative pressure should be felt inside the respirator. If any leakage is detected, adjust position of respirator and/or tension of strap. Retest the seal. Repeat the procedure until the respirator is sealed properly.

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